


# Multilink® Automix

Placement of glass ceramic restorations  
(Inlays, onlays, crowns, bridges)

Befestigung von Glaskeramik-Restaurationen  
(Inlays, Onlays, Kronen, Brücken)



## Restoration Restauration

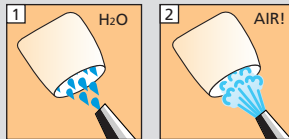

**Pre-treatment of ceramic restorations:** According to manufacturer's instructions. e.g.

Vorbehandlung der Keramik: Nach Angaben des Herstellers, z.B.


IPS Empress: Etch for 60 sec

IPS e.max: Etch for 20 sec


with IPS Ceramic etching gel (hydrofluoric acid). Ätzen mit IPS Ceramic Ätzgel (Flussäure).

Follow safety instructions! Sicherheitshinweise beachten!



## Monobond-S

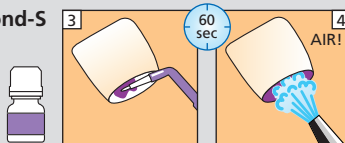

## Dentin/Enamel Dentin/Schmelz

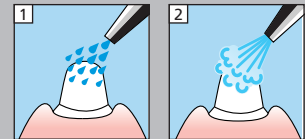


**Mix Multilink  
Primer A+B 1:1**








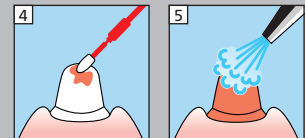
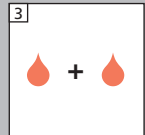


15 sec application


15 Sek. applizieren

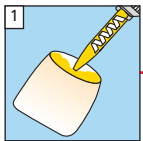

Apply on restoration  
Auftragen auf  
Restauration

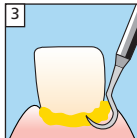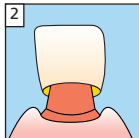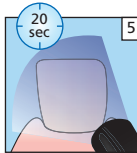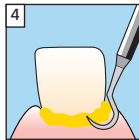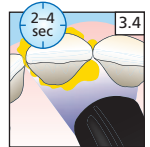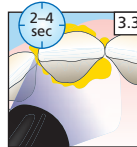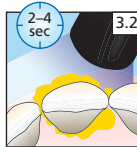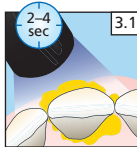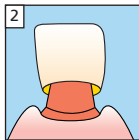

120  
sec

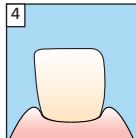

Remove excess immediately!  
Überschüsse sofort entfernen!

CE 0123

Please read Instructions for Use  
Bitte lesen Sie die Gebrauchsinformation

ivoclar  
vivadent  
clinical

616359/1108/d/e/BVD