Instructions for Use
Gebrauchsinformation
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Instruções de Uso
Bruksanvisning
Brugsanvisning
Käyttöohjeet
Productinformatie
Οδηγίες Χρήσεως
Kullanım Kılavuzu

- Anatomically shaped rubber dam
- Anatomisch ausgeformter Kofferdam
- Digue anatomique
- Diga di gomma di forma anatomica
- Dique de goma con forma anatómica
- Dique de borracha com conformação anatómica
- Anatomisk utformad kofferdam
- Anatomisk udformet kofferdam
- Anatomisesti muotoiltu kofferdam
- Anatomisk formet kofferdam
- Anatomisch gevormde cofferdam
- Ανατομικά διαμορφωμένος ελαστικός απομονωτήρας
- Anatomik olarak şekillendirilmiş rubber dam

For dental use only!

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein
Ivoclar Vivadent AG, FL-9494 Schaan

CE 0123

ivoclar
vivadent®
clinical

Instructions for Use

Description

OptraDam is an anatomically shaped rubber dam for the effective isolation of the working field. It provides retraction of lips and cheeks during dental treatment and offers the clinician increased visibility and accessibility. In addition, the device facilitates the evacuation of water spray. OptraDam features high flexibility and elasticity in all directions of movement, provides patient comfort and assists in keeping the patient's mouth open. OptraDam is made of latex and powdered with corn starch. OptraDam is available in regular and small sizes.

Indication

Suitable for use in therapeutic dental procedures (in particular restorative procedures and endodontic treatment).

Contraindication

The use of OptraDam is contraindicated for patients with latex allergy.

Description of the procedure

When OptraDam is used, the small inner ring of the device is positioned in the area of the gingivobuccal fold, while the outer ring remains outside the mouth. The elastic component between the two rings thus embraces the lips of the patient and provides retraction due to the restoring force of the rings (see Figure 1).

Fig 1: OptraDam in place

1. Select the appropriate sized OptraDam. The size (R for regular or S for small) is indicated on the back of the latex bag.
2. The rubber dam punch is used to make the appropriate holes, depending on the teeth to be exposed.
3. OptraDam may be inserted from both the 12 o'clock position with the patient lying down, or from the 8 o'clock position with the patient sitting upright (or, the 4 o'clock position if the clinician is left-handed).
4. To ease placement, the operator should grasp the inner ring by reaching through the outer ring. The inner ring should be held between the thumb and index finger and pressed together slightly (see Figure 2).

Fig 2: Correct hold on the inner ring with thumb and index finger. The ring is slightly compressed in the process.

5. The slightly compressed ring is then inserted into the buccal corridor on one side, so that the elastic component embraces the corner of the mouth and the outer ring is still located extraorally (see Figure 2).
6. As soon as OptraDam is securely in place on one side, the other side is inserted in the same manner by slightly bending it (see Figure 2).

Fig 3: After having positioned the inner ring behind the corners of the mouth, it is pushed behind the upper and lower lip.

7. Subsequently, the inner ring is placed behind the upper and lower lips and thus the OptraDam assumes its final, stable position (see Figure 3).
8. The dam is secured around the teeth by fitting the septum of the dam interproximally and in the sulcus using dental floss.
9. The securely positioned OptraDam improves the overall view and accessibility of the working field. The saliva ejector is introduced below the integrated frame and positioned in the corner of the mouth. Lateral movements of the mandible and opening of the mouth to various degrees may provide additional space and access to the treatment area. If the interproximal spaces are large and retention of the rubber dam is insufficient, a ligature of dental floss or Wedjets may be used additionally to hold the latex bag in place.
10. Before the dam is removed, each septum is carefully cut. For the removal of OptraDam, the outer rim is held and compressed slightly, so that the inner ring is loosened from the gingivobuccal fold. The inner ring is carefully removed from one corner of the mouth. Due to its restoring force it will then slip free on its own.

If, in individual cases, the inner ring should show a tendency to slip out of the mouth in the area of the lower lip when the mouth is closed completely, positioning of the inner ring deeper inside the vestibule is usually sufficient to solve the problem. Sometimes, the use of the other size may also be a solution.

Note

OptraDam is for single patient use only.

In patients with impeded respiration through the nose or for suctioning purposes, a hole may be cut into the latex bag in the palatal region using scissors. As OptraDam contains natural rubber latex, it may provoke an allergic reaction in sensitive individuals.

Keep out of the reach of children!

For use in dentistry only!

Date of preparation: 05/2005

Revision 0

Manufacturer:

Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent AB
Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.