OptraContact

Instructions for Use
Gebrauchsinformation
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Instruções de Uso
Bruksanvisning
Brugsanvisning
Käyttöohjeet
Bruksanvisning
Productinformatie
Οδηγίες Χρήσεως

- Contact-point instrument
- Kontaktpunkt-Instrument
- Instrument pour point de contact
- Strumento per la realizzazione dei punti di contatto
- Instrumento para puntos de contacto
- Instrumento para ponto de contacto
- Kontaktpunkt-instrument
- Kontaktpisteinstrumentti
- Kontaktpunktinstrument
- Contactpuntinstrument
- Εργαλείο σημείων επαφής

For dental use only.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Germany

Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

ivoclar
vivadent
clinical

583588/0704/25/WE3/G

English

Instructions for Use

Description:

The OptraContact instrument assists dental professionals in achieving tight and properly contoured proximal contacts during the placement of Class II restorations.

With OptraContact, the matrix band is pressed against the adjacent tooth during the light-curing procedure. Since the instrument features a bifurcated end, a composite bridge is formed at the desired point of contact. This bridge helps to stabilize the matrix and greatly facilitates the shaping of a restoration to final contour and the creation of tight and proper proximal contacts.

Indication

While OptraContact Molar (green instrument) is specifically designed for large cavities, OptraContact Premolar (blue instrument) is designed for small Class II restorations.

Contraindications

If the bifurcation instrument is too wide for the preparation (specifically the proximal box), the instrument cannot be used.

Use

- 1) Following cavity preparation, the matrix band is placed and wedged.

- 2) After the bonding agent has been applied, the first proximal increment of composite is placed and adapted to the cavity margins using a suitable instrument.

- 3) Place OptraContact into the composite material along the matrix band and press against the adjacent tooth. The composite material is evenly distributed throughout the cavity, thus creating the contact point.

- 4) With the instrument held in place, light-cure the initial increment of composite according to the manufacturers specification.

- 5) Remove the instrument. A contact point (bridge) of composite has been created.

Notes

- In very deep cavities, it may be advisable to use OptraContact during the application of the second or third composite layer.
- OptraContact is autoclavable.
- OptraContact should not be used if the bifurcated end becomes distorted or roughened. Otherwise, it might be difficult to remove the instrument once the composite has been light-cured.
- The use of excessive force should be avoided when working with the instrument, as additional tightening of the matrix band or the application of direct pressure to the tooth may cause damage to the remaining tooth structure.
- It is important to carefully fill the two depressions that have formed on both sides of the composite bridge with composite material.

Keep out of the reach of children!

For dental use only!

Date information prepared: 05/2004

Revision 0

Manufacturer: Ivoclar Vivadent AG, FL-9494 Schaan/FL

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the materials for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent AB
Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.
Dentigen Inc.