OptraMatrix

Instructions for Use
Gebrauchsinformation
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Instruções de Uso
Bruksanvisning
Brugsanvisning
Käyttöohjeet
Bruksanvisning
Productinformatie
Οδηγίες Χρήσεως

- Partially thinned circular matrix band
- Gezielt ausgedünntes Zirkulärmatrizenband
- Matrice circulaire à zones de contact sélectives plus fines, et perforée sur la face opposée
- Matrice circolare a banda sottile
- Banda matriz circular parcialmente adelgazada
- Banda parcialmente afilada para matriz circular
- Rundformade matrisesband med uttunnad sektion
- Cirkulært matricebånd med rude
- Osittain ohennettu hampaan ympäri vietävä matrisinauha
- Ekstra tynt sirkulært matrisebånd
- Geedeeltelijk verdunde ronde matrixband
- Κυκλικό τεχνητό τοίχωμα μερικώς μικρού πάχους

For dental use only.
Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Germany
Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

590875/0704/25/WE3/G

ivoclar
vivadent®
clinical

English

Instructions for Use

Description

OptraMatrix has been specially designed to facilitate the contouring of direct multi-surface restorations in the posterior region (Class II). The selectively thinned contact areas of the matrix (thinned to 10 micrometer) and the window for the intact contact on the uninvolved side (in two-surface restorations) assist in creating precise, anatomically shaped contact points of consistent tightness. The window and the selectively thinned contact area ensure that only a thin layer of foil separates the adjacent teeth during the restorative procedure. The need for forceful separation of the teeth is thus eliminated.

Indication

Suitable for single use in direct Class II restorations in conjunction with composite filling materials.

Shapes

Two different shapes are available for two-surface restorations. The shapes are selected according to the position of the proximal box and the quadrant in question. The universal MOD shape is used for preparations that involve three surfaces.

On the reverse side of the matrix band, the following schemes can be found:

Legend

Quadrant I	Quadrant II
Quadrant IV	Quadrant III

Or,

Teeth (1 – 8)	Teeth (9 – 16)
Teeth (25 – 32)	Teeth (17 – 24)

M – mesial

D – distal

1. OptraMatrix Molar MOD / Universal

three surfaces – all quadrants
(The gingival extensions may be trimmed using scissors, if required)

2. OptraMatrix Molar


Or,

I M	II D
IV D	III M
(1 – 8) M	(9 – 16) D
(25 – 32) D	(17 – 24) M

(The gingival extensions may be trimmed using scissors, if required)

3. OptraMatrix Molar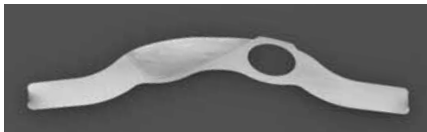

Or,

I D	II M
IV M	III D
(1 – 8) D	(9 – 16) M
(25 – 32) M	(17 – 24) D

(The gingival extensions may be trimmed using scissors, if required)

4. OptraMatrix Premolar MOD/Universal

three surfaces – all quadrants
(May also be suitable for small preparations in molars, or in pedo applications)

5. OptraMatrix Premolar


Or,

I M	II D
IV D	III M
(1 – 8) M	(9 – 16) D
(25 – 32) D	(17 – 24) M

(May also be suitable for small preparations in molars, or in pedo applications)

6. OptraMatrix Premolar

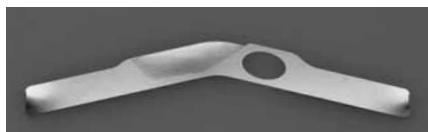Or,

I D	II M
IV M	III D
(1 – 8) D	(9 – 16) M
(25 – 32) M	(17 – 24) D

(May also be suitable for small preparations in molars, or in pedo applications)

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Ivoclar Vivadent AB
Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.
Dentigenix Inc.