OptraPol

Instructions for Use Gebrauchsinformation Mode d'emploi Istruzioni d'uso Instrucciones de uso Instruções de Uso Bruksanvisning Brugsanvisning Käyttöohjeet Productinformatie Οδηγίες Χρήσεως Kullanım Kılavuzu

- High-gloss polishing system for composite materials
- Hochglanz-Poliersystem für Composites
- Système de polissage pour les matériaux composites
- Sistema di lucidatura per materiali compositi
- Sistema de pulido a alto brillo para materiales composite
- Sistema de polimento de alto brilho para compósitos.
- System för höggjanspolering av kompositmaterial
- Høygjanspoleringsystem til kompositter
- High-gloss kiillotusjärjestelmä yhdistelmämuoville
- Høygjanspoleringsystem for kompositter
- Hoogglanspolijststelsysteem voor composieten
- Σύστημα για την υψηλής στιλπνότητας στίλβωση σύνθετων υλικών
- Komposit materyalleri yüksek derecede parlatma sistemi

For dental use only!

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein
Ivoclar Vivadent AG, FL-9494 Schaan

CE 0123

ivoclar
vivadent®
clinical

English

Instructions for Use

Description

OptraPol is a multiple-use, one-step high-gloss polishing system for composite materials that consists of four differently shaped instruments.

Shapes

- Small flame
- Large flame
- Cup
- Disc

Composition

The polishers consist of caoutchouc, silicone carbide, aluminium oxide, titanium oxide, and iron oxide. The handles are made of stainless steel.

Indication

OptraPol is used for polishing direct tooth-coloured restorations to a high gloss.

Contraindication

If a patient is known to be allergic to any of the above components, the polishers should not be used.

Side effects

None known to date.

Application:

Direct high-gloss polishing is accomplished in a single step, exerting moderate pressure.

Important notes

- Recommended speed: 10,000–20,000 rpm
- Only use in conjunction with copious water spray.
- Avoid excess heat build-up.
- OptraPol is autoclavable up to 135 °C / 275 °F.
- The use of polishing pastes is not recommended.

Warnings

Do not inhale grinding dust.

Storage

- Store OptraPol at 2–28 °C (26–82 °F)
- Shelf life: see date of expiration
- Do not use OptraPol after the indicated date of expiration.

Store out of the reach of children.

For use in dentistry only.

Date of preparation: 05/2005

Revision 0

Ivoclar Vivadent AG
FL-9494 Schaan/Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent AB
Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.