OptraSculpt

Instructions for Use
Gebrauchsinformation
Mode d'emploi
Istruzioni d'uso
Instrucciones de uso
Instruções de Uso
Brugsanvisning
Brugsanvisning
Käyttöohjeet
Brugsanvisning
Productinformatie
Οδηγίες Χρήσεως

- Composite Sculpting & Modelling instrument
- Modellier-Instrument für Composites
- Instrument à sculpter et à modeler les composites
- Strumento di modellazione per compositi
- Instrumento para el Modelado de composites
- Instrumento para modelar e esculpir compositos
- Modelleringinstrument för kompositier
- Modelleringinstrument til komposit
- Yhdistelmämuovien muotoiluinstrumentti
- Modelleringinstrument for kompositter
- Modelleerinstrument voor composieten
- Εργαλείο διαμόρφωσης συνθέτων ρητινών

For dental use only.

Caution: U.S. Federal Law restricts this device to sale by or on the order of a licensed dentist.

Made in Liechtenstein
Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

ivoclar
vivadent
clinical

590853/0704/WE3/G

English

Instructions for Use

Description

The OptraSculpt system consists of two instruments and disposable elastic modeling tips in six different shapes. OptraSculpt allows easy application and manipulation of all composite materials due to the low adhesion of the modelling tips to composites.

Shapes

Shapes of the handles

Modelling tips

Recommended applications:

- 1 Cylinder**
Initial increments of Class I & II restorations
- 2 Chisel**
Marginal ridges, cervical margins of Class V fillings
- 3 Spatula**
Class V fillings, all facial and buccal surfaces (or smooth surfaces in general). Also recommended for the removal of excess cement
- 4 Ball**
Developing concavity of the lingual surface on anterior restorations.

Pyramid

It is recommended to use the pyramid-shaped tips to accentuate fissures (Class I and II) and to create detailed morphology.

Notes

- The OptraSculpt handle can be autoclaved.
- The modelling tips are intended for single use and must be removed prior to sterilization in an autoclave.
- As small parts present a potential aspiration hazard, the use of rubber dam is recommended.

**Keep out of the reach of children!
For use in dentistry only!**

**Date information prepared: 05/2004
Revision 0**

Manufacturer:
voclar Vivadent AG, FL-9494 Schaan/Liechtenstein

This material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the materials for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent AB
Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.
Dentigenix Inc.