Harvest Purity Wax™

Milling wax for use in CNC
 Cire de fraiseage pour utilisation dans le CNC
 Fräswachs zur Verarbeitung in dem CNC
 Cera de fresado para el procesamiento en la fresadora CNC

Harvest Dental Products, LLC
 155 Arovista Circle, Suite B
 Brea, CA 92821 USA
 Tel: 1+714-674-7400
 Fax: 1+714-674-7402
 Email: hello@harvestdental.com
 Web: www.harvestdental.com

INSTRUCTIONS FOR USE – ENGLISH
INTENDED USE Harvest Purity Wax™ - 100 % burn-out wax for the manufacture of inlays, onlays and single crowns to large bridges for cast metal and pressing technique.
MILLING PARAMETERS _DRILL SPEED: 6,000 – 9,000 RPM _FEED RATE: 9MM/ SEC _BURS: PMMA or ZIRCONIA BURS (NO MORE THAN 2 FLUTE)
COMPOSITION _Proprietary Paraffin wax blend. _Color pigments.
SHADE Colored
HAZARD WARNINGS: When processing, please use personal protective clothing (dust mask, protective goggles).
STORAGE TIP: Refrigerate blanks before milling for extra firmness.
INDICATIONS _ Can be cast, pressed or over-pressed. _ Not a clinical dental product; not suitable for use in the oral cavity.
WORKING INSTRUCTIONS _Follow the manufacturer’s instructions for materials in which the substructure is cast or pressed. Ensure that objected is dust-free.
GUARANTEE Application recommendations, no matter whether verbally, in writing, or by means of practical instructions are based on our experience and tests and are thus only a recommendation. Our products are subject to continuous further development. We reserve the right to alter the products in terms of design or composition.

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