ivoclar vivadent

Recommended for inlays, onlays, 3/4 crowns, telescopes, crown crowns, posts, short and long span bridges, PFM crowns, implant superstructures, partial dentures.

Indications

Composition

Au	40.0
Pt	39.9
Ag	9.7
Co	1.0
Sn	0.2
Zn	0.1
Mn	0.05
Fe	0.05
Si	0.05
Al	0.05
Other	0.05

Evolution® Lite

White, gold-based dental ceramic casting alloy, Type 4

Instructions for Use
Instrucciones de uso
Mode d'emploi
Gebruiksaanvisning
Brugsanvisning
Объятег Xpъoвaw
Käyttöohjeet

Evolution® Lite

White, gold-based dental ceramic casting alloy, Type 4

<p>ivoclar Vivadent Worldwide</p> <p>Brazil Ivoclar Vivadent Ltda. Rua Geraldo Flausino Gomes, 78 - 6.º andar Cjs. 61/62 Bairro: Brooklin Novo CEP: 04575-060 - São Paulo - SP Tel: +55 (11) 3466 0800 Fax: +55 (11) 3466 0840 www.ivoclarvivadent.com.br</p> <p>Canada Ivoclar Vivadent Inc. 2785 Skyway Avenue, Unit 1 Mississauga, Ontario L4W4Y3 Tel: +1 905 238 5700 Fax: +1 905 238 5711 www.ivoclarvivadent.ca</p> <p>China Ivoclar Vivadent Marketing Ltd. Rm 603 Kuen Tang International Business Plaza No. 798 Zhao Jia Bang Road Shanghai 200030 Tel: +86 21 5456 0776 Fax: +86 21 6445 1561 www.ivoclarvivadent.com.cn</p> <p>Colombia Ivoclar Vivadent Marketing Ltd. Calle 134 No. 7-8-83, Of. 520 Bogotá Tel: +57 1 627 33 99 Fax: +57 1 633 16 63 www.ivoclarvivadent.com</p> <p>France Ivoclar Vivadent SAS B.P. 118 F-73410 Saint-Jorioz Tel: +33 450 88 64 00 Fax: +33 450 88 64 00 www.ivoclarvivadent.fr</p> <p>Austria Ivoclar Vivadent GmbH Brenschtr. 16 Postfach 223 A-6706 Burs Tel: +43 5552 624 49 www.ivoclarvivadent.com</p> <p>USA Ivoclar Vivadent Inc. 175 Pineview Drive Amherst, NY 14228 Tel: +1 800 533 6825 Fax: +1 716 691 2285 www.ivoclarvivadent.us</p> <p>Germany Ivoclar Vivadent GmbH Dr. Adolf-Schneider-Str. 2 D-73479 Ellwangen Tel: +49 (0) 79 61 / 8 89-0 Fax: +49 (0) 79 61 / 63 26 www.ivoclarvivadent.de</p> <p>UK Ivoclar Vivadent UK Limited Ground Floor Compass Building Feldspar Close Warrens Business Park Enderby Leicester LE19 4SE England Tel: +44 116 284 78 80 Fax: +44 116 284 78 81 www.ivoclarvivadent.co.uk</p> <p>Russia Ivoclar Vivadent Marketing Ltd. Derbenevskaja Nabereznaja 11, Geb. W 115114 Moscow Tel: +7 495 913 66 19 Fax: +7 495 913 66 15 www.ivoclarvivadent.ru</p> <p>Singapore Ivoclar Vivadent Marketing Ltd. 171 Chin Swee Road #02-01 San Centre Singapore 169877 Tel: +65 6535 6775 Fax: +65 6535 4991</p> <p>Spain Ivoclar Vivadent S.A. E-28037 Madrid Tel: +34 913 75 78 20 Fax: +34 913 75 78 38 www.ivoclarvivadent.com</p> <p>Sweden Ivoclar Vivadent AB Dalsvägen 14 S-169 56 Solna Tel: +46 (0) 8 514 93 930 Fax: +46 (0) 8 514 93 940 www.ivoclarvivadent.se</p> <p>Turkey Ivoclar Vivadent Liaison Office Ahi Evran Caddesi no 1 Polaris Is Merkezi Kat: 7 80670 Maslak Istanbul Tel: +90 536 577 12 62</p> <p>UK Ivoclar Vivadent UK Limited Ground Floor Compass Building Feldspar Close Warrens Business Park Enderby Leicester LE19 4SE England Tel: +44 116 284 78 80 Fax: +44 116 284 78 81 www.ivoclarvivadent.co.uk</p> <p>Mexico Ivoclar Vivadent S.A. de C.V. Av. Mazatlan No. 61, Piso 2 Col. Condesa 06170 México City, D.F. Bogotá Tel: +52 (55) 50 62 10 29 Fax: +52 (55) 50 62 10 29 www.ivoclarvivadent.com.mx</p> <p>New Zealand Ivoclar Vivadent Ltd 12 Omega St, Albany PO Box 5243 Wellesley St Auckland Tel: +64 9 630 52 06 Fax: +64 9 914 99 90</p>	<p>Rev. Date 11/10 Rev. 9</p> <p>www.ivoclarvivadent.com</p> <p>Caution: US Federal Law restricts this device to sale by or on the order of a licensed dentist.</p> <p>LOT</p> <p>ivoclar vivadent passion vision innovation</p>
--	---

EN INSTRUCTIONS FOR USE

MODELLATION
Design the framework in a reduced anatomical shape taking the planned veneer into consideration. Single crowns require a thickness of minimum 0.3 mm; abutment crowns thickness a minimum of 0.5 mm. Make sure the framework demonstrates adequate stability of shape. Avoid sharp angles. Design the connector areas to be adequate for the position and alloy being used.

SPRUNG
Provide the modeled bridge framework or coping with sprues of a suitable size. Use the direct or indirect technique being sure that the reservoir is positioned in the heat center. The connection sprues between the reservoir and the coping should be 2.5-3.0 mm in length and width.

INVESTMENT
Weigh the wax pattern including the sprue to determine the quantity of the alloy to be used. (See wax conversion sheet/formula: weight x density = gr. of alloy). Use investment following the manufacturer's instruction.

BURN-OUT
The suggested burnout temperature: 750-820C/1380-1510F

MELTING AND CASTING
Use a separate ceramic crucible for each alloy and preheat the crucible in the burnout furnace. Used and new alloys must be in a ratio of 1:1. Depending on the type of casting machine, follow the manufacturer's instructions for use. If you use Ivoclar Vivadent Magic Wand, set propane to 0.35 bar/5 psi and oxygen to 0.7 bar/10 psi pressure. Keep the alloy in the reducing atmosphere of the flame between the inner and outer cones. Do not use flux. After casting bench cool to room temperature.

Casting Temperature: 1315-1375C/2400-2510F

METAL PREPARATION
Carefully divest and clean the object with Al₂O₃. Do not use a hammer for divesting the object to prevent deformation. Finish the framework with carbide burs or with ceramic-bonded grinding instruments. Avoid inhalation of dust during grinding!

OXIDATION
Blast surface with 50-100 micron Al₂O₃ at max. 5.2 bar/75 psi pressure before oxidation. Subsequently, clean with ultrasonic or steam. Place the object on the firing tray and provide adequate support. Place the tray in a porcelain furnace set at a low temperature of 650C/1200F and increase the furnace temperature to 950C/1740F without vacuum with 5 min. hold time at top temperature.

SOLDERS AND FLUXES
Design the soldering patty as small as possible and preheat it in the furnace at approximately 600C/1112F. The soldering gap should be the same thickness as the soldering strip. Allow the object to cool slowly after soldering.

Pre-Solder: HGPKF 1015 Y, High Fusing Bondal Flux, HGPKF 1030 Y
Post-Solder: Low Fusing White Gold Solder, .585 Fine Solder, Bondal Flux
Laser weld material: Laser Ceramic White

POLISHING
After glazing remove oxide and flux residue and finish/polish the framework with rubber finishers/polishers.

INDICATIONS
Recommended for inlays, onlays, 3/4 crowns, telescopes, crown crowns, posts, short and long span bridges, PFM crowns, implant superstructures, partial dentures.

CONTRAINDICATIONS
For patients with known allergy/sensitivity to any major or minor constituents of this alloy, consultation with a physician is recommended.

SIDE EFFECTS
In individual cases, sensitivity or allergies to components of this alloy may occur.

INTERACTIONS
Galvanic effects may occur between different alloys in the same oral environment.
For additional information look into the alloy property chart.

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