

## MultiCore® Flow automix syringe – at a glance



MultiCore is a self-curing, fluoride-containing, radiopaque, highly filled composite with light curing option. It demonstrates excellent mechanical properties for core build-up restorations of vital and non-vital teeth.

The excellent MultiCore Flow is **now** also available in the 10g automix syringe and comes in shades light (A1/B1), medium (A2/A3) and white.

### Main sales arguments

#### ***MultiCore is available in two consistencies: Heavy Body and Flow***

The right choice for every technique:

The high-viscosity MultiCore HB is particularly suitable for individually moulded composite cores. The material can be conveniently applied and moulded using suitable instruments. For dental professionals who prefer to use matrices when fabricating core build-ups, MultiCore flow is the material of choice as it offers a flowable consistency. It adapts very well to the dentin surfaces after applying the bonding agent. Thus a reliable bond between the core build-up material and the remaining tooth structure can be achieved. It can also be used for the luting of glass fibre reinforced posts.

#### ***MultiCore is the composite core build-up material from the Tetric company Ivoclar Vivadent***

The convincing physical properties of MultiCore, such as hardness, flexural strength and bond strength, reflect the company's long-term experience in the field of composite technology.

#### ***MultiCore is convenient and practical to use***

Due to the automix delivery system, MultiCore Flow can be quickly mixed and precisely applied using the intra-oral tips provided.

MultiCore HB does not stick to the instrument and can be conveniently shaped and moulded. MultiCore is a self-curing composite. Light-curing reduces the curing time. Thus the material may be placed in one increment and individual control of the curing process is possible.

<b>Delivery form and contents</b>	<b>Article No</b>
<b>MultiCore Flow Refill 10g, light</b>	<b>604166AN</b>
<b>MultiCore Flow Refill 10g, medium</b>	<b>604167AN</b>
<b>MultiCore Flow Refill 10g, white</b>	<b>604168AN</b>
1 x Syringe MultiCore Flow 10g 10 x Mixing Tips small 10 x Intra Oral Tips small 1 x Instructions for Use MultiCore	
<b>MultiCore Intra Oral Tips small Refill /10</b>	<b>604210</b>
10 Intra Oral Tips small	
<b>MultiCore Mixing Tips small Refill /10</b>	<b>604209</b>
10 Mixing Tips small	
<b>MultiCore Flow Intro Pack, light</b>	<b>578910AN</b>
<b>MultiCore Flow Intro Pack, medium</b>	<b>578911AN</b>
<b>MultiCore Flow Intro Pack, blue</b>	<b>578912AN</b>
1 x Cartridge MultiCore Flow 50g 1 x Manual Dispenser 25ml 1:1 20 x Mixing Tips 20 x Intra Oral Tips 1 x AdheSE Primer 2.5g 1 x AdheSE Bonding 2.5g 1 x AdheSE pad with cover 25 x AdheSE applicator, green 25 x AdheSE applicator, black 1 x Flow Chart MultiCore 1 x Flow Chart MultiCore Flow 1 x Instructions for Use MultiCore 1 x Instructions for Use AdheSE	
<b>MultiCore Flow Refill 50g, light</b>	<b>578913AN</b>
<b>MultiCore Flow Refill 50g, medium</b>	<b>578914AN</b>
<b>MultiCore Flow Refill 50g, blue</b>	<b>578915AN</b>
1 x Cartridge MultiCore Flow 50g 20 x Mixing Tips 20 x Intra Oral Tips 1 x Flow Chart MultiCore 1 x Flow Chart MultiCore Flow 1 x Instructions for Use MultiCore	
<b>MultiCore Dispenser Refill</b>	<b>579094</b>
1 x Dispenser 25 ml 1:1	
<b>MultiCore Intra Oral Tips Refill /50</b>	<b>579095</b>
50 Intra Oral Tips, yellow	
<b>MultiCore Mixing Tips Refill /50</b>	<b>579096</b>
50 Mixing Tips, blue	

<b>Delivery form and contents</b>	<b>Article No</b>
<b>MultiCore HB Intro Pack, light</b>	<b>578916AN</b>
<b>MultiCore HB Intro Pack, medium</b>	<b>578917AN</b>
<b>MultiCore HB Intro Pack, blue</b>	<b>578918AN</b>
2 x Syringes MultiCore HB Base 4g 2 x Syringes MultiCore HB Catalyst 4g 1 x Polyart mixing pad 50x70 mm 5 x Mixing spatula, plastic 1 x AdheSE Primer 2.5g 1 x AdheSE Bonding 2.5g 1 x AdheSE pad with cover 25 x AdheSE applicator, green 25 x AdheSE applicator, black 1 x Flow Chart MultiCore 1 x Instructions for Use MultiCore 1 x Instructions for Use AdheSE	
<b>MultiCore HB Refill 4 x 4g, light</b>	<b>578919AN</b>
<b>MultiCore HB Refill 4 x 4g, medium</b>	<b>578920AN</b>
<b>MultiCore HB Refill 4 x 4g, blue</b>	<b>578921AN</b>
2 x Syringes MultiCore HB Base 4g 2 x Syringes MultiCore HB Catalyst 4g 1 x Polyart mixing pad 50x70 mm 5 x Mixing spatula, plastic 1 x Flow Chart MultiCore 1 x Instructions for Use MultiCore	