

Programat®

Instructions for the software update via USB stick (P310, P510, P710, EP 3010, EP 5010, CS2, CS3):

Software updates can be easily installed on the furnace by means of a USB stick. For the purpose, a USB memory stick is required which contains a current software file (e.g. P510:V1.10.iv). The software version on the USB memory stick must be higher than the one installed on the furnace.

The free software updates for Programat furnaces are available from www.ivoclarvivadent.com/downloadcenter.

A software update requires only a few steps:

1. Open the software update

Press the **[Software update]** button in the home screen.



2. Conduct the software update




If the USB stick with the software file is already connected, the furnace automatically searches for a valid software file. If the USB stick has not yet been connected with the furnace, do so now. Press the **[Execute]** button.

3. The status bar shows the progress of the update.

4. Complete the software update

The following messages are displayed:

-  Software update successful
-  Software update failed

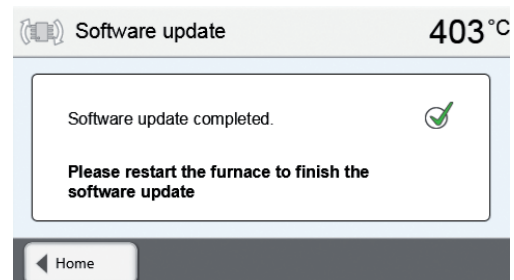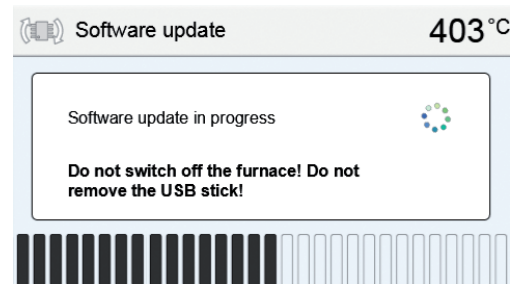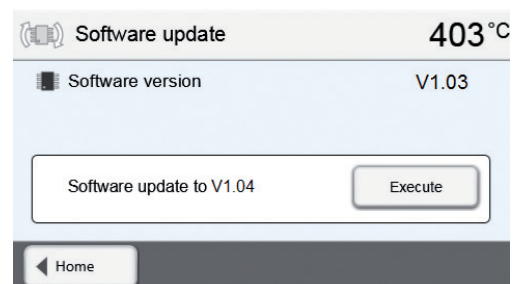

The furnace has to be switched off and on again by means of the mains switch at the rear of the furnace to complete the software update.



Important information

Please note that modified Ivoclar Vivadent programs may be overwritten during a software update. Individual programs are not affected and will not be overwritten.