


Tetric® Prime

1


Self-Etch



Etch & Rinse

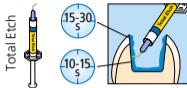


Selective Enamel-Etch

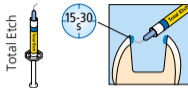

2

Adhese® Universal
VivaPen®

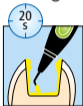

Bluephase®
PowerCure

3

Tetric®
EvoFlow®



or

Tetric®
PowerFlow







4

Tetric® Prime



mW/cm ²	Tetric Prime/ Tetric EvoFlow	Tetric PowerFill/ Tetric PowerFlow	Adhese Universal
500 – 900	20 s	20 s	10 s
900 – 1400	10 s	10 s	10 s
1800 – 2200	5 s	5 s	–
Direct posterior restoration, light-cured from occlusal aspect Direkte Seitenzahnrestauration, okklusale Belichtung			
2700 – 3300	–	3 s	3 s

Tetric® Prime

	Conventional composites				4-mm-composites 3s	
	FLOWABLE		SCULPTABLE		FLOWABLE	SCULPTABLE
	Tetric EvoFlow®		Tetric® Prime		Tetric® PowerFlow	Tetric® PowerFill
						
	Dentin & Bleach XL	Enamel	Dentin	Enamel	Dentin-like translucency	Enamel-like translucency
Layer thickness up to	1.5 mm	2 mm	1.5 mm	2 mm	4 mm	4 mm
Exposure Time ≥ 500 mW/cm ²	20 s				20 s	
900 – 1 400 mW/cm ²	10 s				10 s	
1 800 – 2 200 mW/cm ²	5 s				5 s	
2 700 – 3 300 mW/cm ² , e.g. Bluephase® PowerCure	–				3 s	



Date information prepared: 2019-06-03/708819/Rev.0
Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

CE 0123

**ivoclar
vivadent**
clinical