

STAINING TECHNIQUE



Fill the restoration with **IPS Object Fix Putty/Flow** and press the **IPS e.max[®] CAD Crystallization Pin** into the putty material.



Apply individual characterizations using **IPS e.max CAD Crystall./Shades and Stains**.



Spray an even and covering layer of **IPS e.max CAD Crystall./Glaze Spray** onto the restoration.



Finished IPS e.max CAD restoration



Smooth out the attachment point. Take proximal contacts and minimum thickness into account.



Conduct the **crystallization firing** using e.g. **Programat[®] CS2**.

SELF GLAZE



Prepolish using e.g. **OptraFine[®] F**. Overheating of the glass-ceramic must be avoided.



High-gloss polishing using e.g. **OptraFine P**. Overheating of the glass-ceramic must be avoided.



Fill the restoration with **IPS Object Fix Putty/Flow** and press the **IPS e.max[®] CAD Crystallization Pin** into the putty material.



Finished IPS e.max CAD restoration